



Vascular Access System

Patient
Information

“I had no surgery and have no scar.”

“I am so happy to hear that Ellipsys has received FDA approval so other patients like me can benefit from this type of fistula. I had no surgery and have no scar. Amazing when I compare my AVF with others in the dialysis unit. There is no comparison.”

~Theresa Torres, Tempe AZ

“I have a family member on dialysis and his arm is all lumpy and bumpy, not mine.”

My arm looks great! After my procedure, I had a bandage placed and my daughter took me to lunch.”

~Paulette, Richmond, VA



For more information,
visit www.bandaidfistula.com


Finally, hemodialysis access that's minimally invasive and patient friendly.

Skip surgery. Go directly to vascular access.

The Ellipsys Vascular Access System is a brand new way to create a percutaneous arteriovenous fistula (AVF) for hemodialysis patients. Avenu Medical has turned a complex surgery into a simple, outpatient procedure that leaves the AVF site vasculature undisturbed. Translation?


Patients everywhere can take advantage of a unique, non-surgical option leading to faster, more efficient, less invasive vascular access. Following this simple outpatient procedure, the patient goes home with just a band aid!

The Ellipsys procedure.

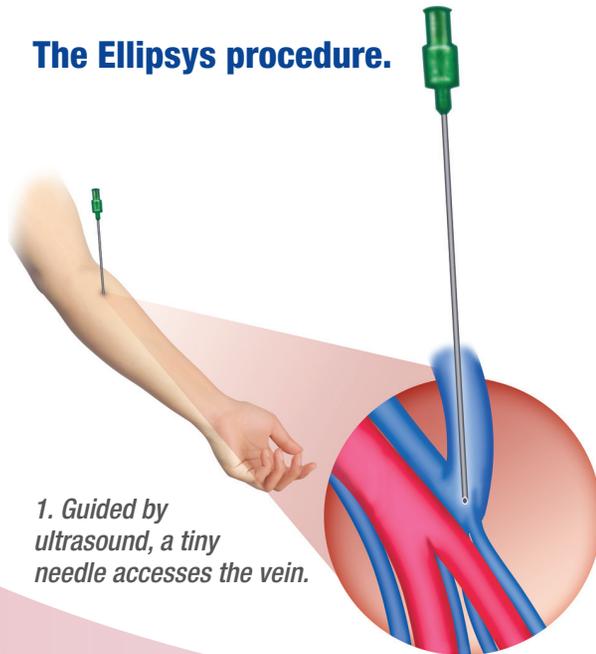


1. Guided by ultrasound, a tiny needle accesses the vein.


2. Once the needle reaches the artery, a guide wire is put into place. The needle is removed and a sheath is advanced into the artery (not shown).

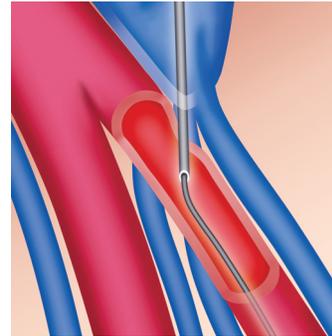


3. Next, the Ellipsys catheter is moved through the sheath, with the catheter tip positioned inside the artery.

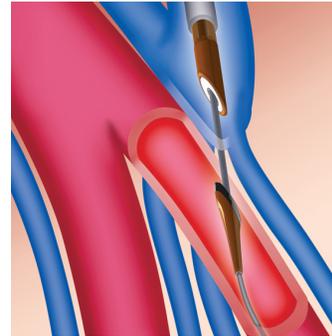

4. The catheter is closed and activated – creating a permanent connection between the vein and artery. Sutures are not required.

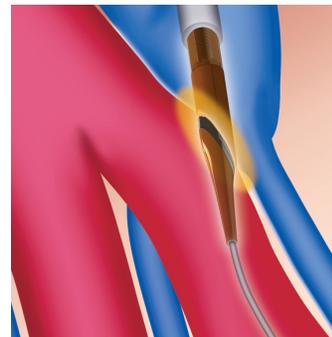

5. Finally, the catheter is removed resulting in AVF flow.

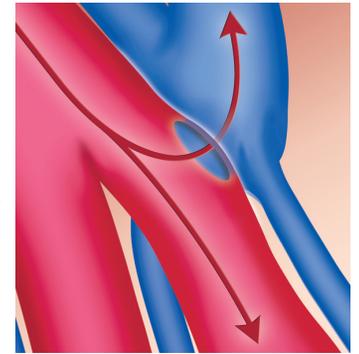

6. Upon completion of the procedure, the patient leaves with just a band aid.



Long-term success.



Day 1:
Patient with
Ellipsys EndoAVF.




18 months:
Patient with
Ellipsys EndoAVF.

Clip and save to present to your dialysis treatment center.



Percutaneous AVF for Hemodialysis Patients
No BP or Needles in this Arm

A new avenue for hemodialysis access.

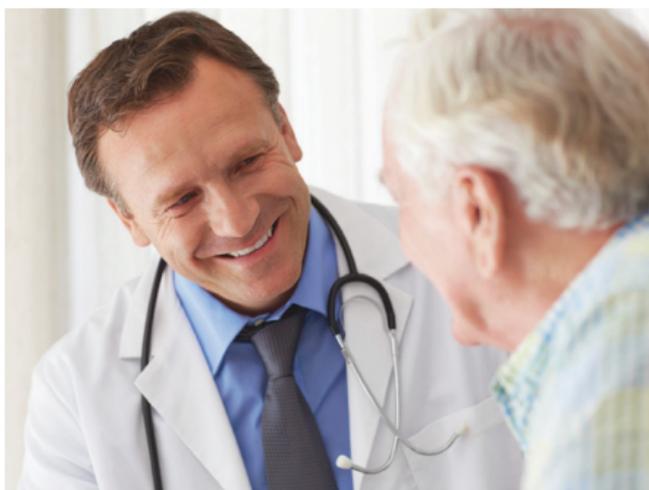

No incisions. No sutures. Our surgery-free, permanent solution is a long-awaited and welcome change for hemodialysis patients. The common surgical procedure for fistula creation has remained the same for half a century. But all that's changed. Unlike surgery, the Ellipsys System uses an endovascular approach ***that leaves the skin scar-free.***



27123 Calle Arroyo, Suite 2101
San Juan Capistrano, CA 92675
Toll Free: 888-EndoAVF (363-6283)

bandaidfistula.com